



2701 Bartram Road • Bristol, PA 19007  
800-840-9041 • 215-788-2700  
www.gentell.com • info@gentell.com

## Bordered Gauze

<b>4" x 4"</b> (2.5"x2.5" pad with 3/4" adhesive border) 100/box <b>GEN-15410</b> NDC# 61554-154-04	<b>6" x 6"</b> (4.5"x4.5" pad with 3/4" adhesive border) 100/box <b>GEN-15610</b> NDC# 61554-156-06
--	--

- Serves as either primary or secondary dressing
- Meets Medicare guidelines for secondary cover dressing
- Gentle, spun-lace, hypoallergenic tape allows skin to breathe and reduces skin tears



**Gentell Bordered Gauze** is a multi-purpose, all-inclusive wound dressing that saves nursing time and is gentle to delicate skin. A non-adherent pad is paired with a conforming beveled spun lace tape to create a secure, comfortable dressing that stays in place.

### Directions

1. Use Gentell Bordered Gauze as cover dressing for any primary or secondary treatment.
2. Flush the wound with Gentell Wound Cleanser and gently dry the skin surrounding the wound site.
3. Apply Gentell Bordered Gauze Dressing. Choose appropriate size based on the measurements of the wound and the size of the pad
4. Remove paper backing from the dressing and apply directly over the surface of the wound.
5. Change dressing daily or as ordered by a physician.