



Calcium Alginate Ag (Silver) Dressing

2701 Bartram Road • Bristol, PA 19007
800-840-9041 • 215-788-2700
www.gentell.com • info@gentell.com

2" x 2" (5 cm. x 5 cm.) 10/carton 5 cartons/case (50 dressings) GEN-13220 NDC# 61554-132-22	4" x 4" (10 cm. x 10 cm.) 10/carton 5 cartons/case (50 dressings) GEN-13520 NDC# 61554-135-24
---	---

- Uniformly absorbs and contains up to 20 times its weight in wound exudates with a sustained release of antimicrobial silver
- Reduces odor primarily caused by bacteria
- Controlled silver release provides antimicrobial protection to the wound



Gentell Calcium Alginate Ag Dressings are a sterile, antimicrobial, comfortable, fiber-structured alginate with high absorbency. These advanced alginate dressings fight bacteria and a broad spectrum of microorganisms while absorbing and containing exudate in a moist healing environment. A steady release of silver and a reaction between the calcium in the dressing and the sodium in the wound exudate creates a gel that fights infection and is easy to remove from the wound.

Directions

1. Gentell's Calcium Alginate with Silver pads should be applied in dry form on shallow wounds including leg ulcers, pressure ulcers, diabetic foot ulcers and surgical wounds. May also be used for minor conditions such as lacerations, abrasions, skin tears and minor burns.
2. Flush the wound with Gentell Wound Cleanser and gently dry the skin surrounding the wound site.
3. Choose appropriately sized 2" x 2" or 4" x 4" dressing.
4. Apply appropriately sized cover dressing such as Gentell Bordered Gauze, Comfortell[®], or Lo-Profile Foam.
5. If used as a secondary dressing, apply directly over primary treatment.
6. Repeat daily or as ordered.