



2701 Bartram Road • Bristol, PA 19007
800-840-9041 • 215-788-2700
www.gentell.com • info@gentell.com

Calcium Alginate Dressing

2" x 2" (5 cm. x 5 cm.) 10/carton. 5catons / case (50 dressings) GEN-13200 NDC# 61554-132-02	4" x 4" (10 cm. x 10 cm.) 10/carton. 5catons. /case (50 dressings) GEN-13500 NDC# 61554-135-04
--	--

- Uniformly absorbs, collects and contains 20 times its weight in exudate
- Can be molded and folded to fit the size of the wound
- Easy to remove – does not adhere to the healing tissue of the wound



Gentell's Calcium Alginate Dressings are a sterile, comfortable, advanced fiber-structured alginate with a highly absorbent capacity. Alginate dressings absorb, collect and contain exudate while providing a moist healing environment. A reaction between the calcium in the dressing and the sodium in the wound exudate creates a gel-like substance that promotes moist wound healing.

Directions

1. Apply Gentell's Calcium Alginate 2" x 2" and 4" x 4" pads in dry form on shallow wounds including leg ulcers, pressure ulcers, diabetic foot ulcers and surgical wounds. May also be used for minor conditions such as lacerations, abrasions, skin tears and minor burns.
2. Flush the wound with Gentell Wound Cleanser and gently dry the skin surrounding the wound site.
3. Choose appropriate sized dressing 2" x 2" or 4" x 4".
4. Apply appropriate size cover dressing using a Gentell Bordered Gauze, Comfortell or Gentell Lo-Profile Foam.
5. If being used as a secondary dressing, apply directly over primary treatment.
6. Change dressing daily or as ordered by a physician.