



2701 Bartram Road • Bristol, PA 19007  
800-840-9041 • 215-788-2700  
www.gentell.com • info@gentell.com

# Dermatell™ Hydrocolloid Wound Dressing

<b>4" x 4"</b> (2" x 2" pad with 3/4" adhesive border) 10/carton 5 cartons /case. (50 dressings) <b>GEN-10200</b> NDC# 61554-102-02	<b>6" x 6"</b> (4.5" x 4.5" pad with 3/4" adhesive border) 10/carton 5 cartons /case] (50 dressings) <b>GEN-10400</b> NDC# 61554-104-00
--	--

- Flexible hydrocolloid wafer absorbs water, forms a gel and promotes moist healing process
- Water resistant border enables normal washing and bathing
- Beveled edge design minimizes lifted edges



**Gentell Dermatell Hydrocolloid Dressing** consists of a hydrocolloid wafer attached to our water resistant adhesive border that is flexible and conforms easily to the body. Soft, pliable Dermatell dressings enhance patient comfort and protection while promoting moist wound healing by drawing out exudates and enabling natural debridement. Dermatell is most effective when kept in place for a minimum of three days.

## Directions

1. Gentell Dermatell should remain on the patient for at least three days and applied on Stage 2 or Stage 3 wounds. **Do not use on infected wounds.**
2. Flush the wound with Gentell Wound Cleanser and gently dry the skin surrounding the wound site.
3. As a primary dressing, apply directly to the wound surface.
4. As a secondary dressing, apply directly over primary treatment.
5. Repeat every three days or as ordered by a physician.