



Hydrogel Ag (silver) Saturated Gauze

2701 Bartram Road • Bristol, PA 19007
800-840-9041 • 215-788-2700
www.gentell.com • info@gentell.com

2" x 2" 12 ply pad 20/carton 6 cartons / case (120 dressings) GEN-11220 NDC# 61554-122-02	4" x 4" 12 ply pad 10/carton 4 cartons / case (40 dressings) GEN-11260 NDC# 61554-112-06
------------------------------------------------------------------------------------------------------------------------	-----------------------------------------------------------------------------------------------------------------------

- Easiest and least expensive way to apply antimicrobial silver sulfadiazine
- Single-use saturated gauze is great for infection control
- Silver inhibits bacteria resistant to other antimicrobial agents without harming healthy tissue



Ingredients for Hydrogel

- Purified Water
- Aloe Vera Extract
- Sorbitol
- Silver Sulfadiazine
- Carbomer
- Triethanolamine
- Propylene Glycol
- Imidazolidinyl Urea
- Methylparaben
- Allantoin

Gentell Hydrogel Ag Saturated Gauze is a 12-ply gauze fully saturated with viscous silver-infused Aloe Vera-based hydrating wound gel. Hydrogel Ag Gauze protects the wound bed and hydrates the healing environment. Silver sulfadiazine decreases the bacterial load of both Gram-positive and Gram-negative bacteria.

Directions

1. Gentell Hydrogel Ag saturated gauze can be used on chronic wounds, pressure ulcers Stage 1-4, first and second degree burns, venous ulcers and dry-to-minimally exudative wounds.
2. Flush the wound with Gentell Wound Cleanser and gently dry the skin surrounding the wound site.
3. To apply, peel open pouch and remove the Hydrogel Ag gauze pad.
4. Following your standard protocol, cover or pack wound loosely with the Hydrogel Ag Gauze.
5. Cover wound with a secondary dressing such as Gentell Bordered Gauze or Gentell Comfortel®.