



2701 Bartram Road • Bristol, PA 19007
800-840-9041 • 215-788-2700
www.gentell.com • info@gentell.com

Hydrogel Ag

4 oz
Tube
12/case
GEN-11240
NDC# 61554-124-04

- Formulated with Silver Sulfadizine, a broad spectrum antimicrobial that disrupts cell walls and membranes of bacteria
- The most cost-effective way to treat a wound with silver
- Inhibits bacteria resistant to other antimicrobial agents without harming healthy tissue



Ingredients

- Purified Water
- Aloe Vera Extract
- Sorbitol
- Silver Sulfadizine
- Carbomer
- Triethanolamine
- Propylene Glycol
- Imidazolidinyl Urea
- Methylparaben
- Allantoin

Directions

1. Gentell Hydrogel Ag can be used on chronic wounds, pressure ulcers Stage 1-4, first and second degree burns, venous ulcers and dry-to-minimally exudative wounds.
2. Flush the wound with Gentell Wound Cleanser and gently dry the skin surrounding the wound site.
3. Apply a 1/8 inch layer of Gentell Hydrogel Ag to the entire surface of the wound using an appropriate clean applicator gauze to cover the wound bed
4. Cover wound with an appropriate secondary dressing like Gentell Bordered Gauze or Gentell Comfortell[®].
5. Change dressing daily or as ordered by a physician.

Gentell Hydrogel Ag is an antimicrobial wound gel that combines the unique properties of silver sulfadizine and our Aloe Vera-based hydrogel. Data suggest that silver sulfadizine decreases the bacterial load of both gram positive and gram negative bacteria.