



2701 Bartram Road • Bristol, PA 19007
800-840-9041 • 215-788-2700
 www.gentell.com • info@gentell.com

Hydrogel Saturated Gauze

2" x 2" 12 ply pad 20/carton, 6 cartons / case (120 dressings) GEN-11200 NDC# 61554-112-02	4" x 4" 12 ply pad 10/carton, 4 cartons / case (40 dressings) GEN-11400 NDC# 61554-114-04
---	--

- Easiest way to apply Hydrogel
- Single use saturated gauze is great for infection control
- Gentell Hydrogel's crystal clear formula makes the wound easier to see and diagnose.



Ingredients for Hydrogel

- Purified Water
- Aloe Vera Extract
- Sorbitol
- Carbomer
- Triethanolamine
- Propylene Glycol
- Imidazolidinyl Urea
- Methylparaben
- Allantoin

Gentell Hydrogel Saturated Gauze is 12-ply gauze fully saturated in crystal clear, viscous Aloe Vera-based hydrating wound gel. Hydrogel gauze protects the wound bed and enhances the environment essential to the healing process. Aloe is also a source of ace mannin, in addition to other mono and polysaccharides, amino acids, glycoproteins, vitamins and enzymes.

Directions

1. Use Gentell Hydrogel on stage 1, 2, 3 and 4 wounds with little or no drainage, diabetic skin ulcers, venous stasis ulcers, first and second degree burns, post-surgical incisions, cuts and abrasions.
2. Flush the wound with Gentell Wound Cleanser, and gently dry the skin surrounding the wound site.
3. To apply, peel open pouch and remove the hydrogel gauze pad.
4. Following your standard protocol; cover or pack wound loosely with the Gentell Saturated Gauze.
5. Cover wound with a secondary dressing such as Gentell Bordered Gauze or Gentell Comfortell®.
6. Change dressing daily or as ordered by a physician.