



2701 Bartram Road • Bristol, PA 19007
800-840-9041 • 215-788-2700
www.gentell.com • info@gentell.com

Hydrogel

4 oz Tube 12/case GEN-11080 NDC# 61554-111-04	8 oz Adjustable Spray Bottle 6/case GEN-11080 NDC# 61554-110-08
---	---

- Hydrates the wound for at least a full 24 hours
- Gentell Hydrogel’s crystal clear formula makes the wound easier to see and diagnose
- Adjustable spray is great for tunneling and hard-to-reach areas



Ingredients

- Purified Water
- Aloe Vera Extract
- Sorbitol
- Carbomer
- Triethanolamine
- Propylene Glycol
- Imidazolidinyl Urea
- Methylparaben
- Allantoin

Gentell Hydrogel is an Aloe Vera-based hydrating wound gel that protects the wound bed. Because it uses less water than other hydrogels, Gentell Hydrogel is more viscous and less “runny.” Aloe is also a source of acemannan, in addition to other mono and poly saccharides, amino acids, glycoproteins, vitamins and enzymes.

Directions

1. Use Gentell Hydrogel on stage 1-4 wounds with little or no drainage, diabetic skin ulcers, venous stasis ulcers, first and second degree burns, post-surgical incisions, cuts and abrasions.
2. Flush the wound with Gentell Wound Cleanser and gently dry the skin surrounding the wound site.
3. Apply a 1/8 inch layer of Gentell Hydrogel dressing to the entire surface of the wound using an appropriate clean applicator or gauze to sufficiently cover the wound bed.
4. Cover wound with a secondary dressing like Gentell Bordered Gauze or Gentell Comfortell®.
5. Change dressing daily or as ordered by a physician.