




2701 Bartram Road • Bristol, PA 19007
800-840-9041 • 215-788-2700
www.gentell.com • info@gentell.com

Gentell Lo-Profile Foam Plus

4" x 4" (2.5"x2.5" pad with 3/4" adhesive border) 10/carton 5 cartons / case (50 dressings) GEN-14400 NDC# 61554-144-00	6" x 6" (sterile) (4.5"x4.5" pad with 3/4" adhesive border) 10/carton 5 cartons / case (50 dressings) GEN-14600 NDC# 61554-146-06
---	---

- Absorbs twenty times its weight in exudates and drainage with no strike through
- Water-resistant bacterial barrier provides incontinent protection
- Excellent pressure relief and protection for skin that has recently healed.

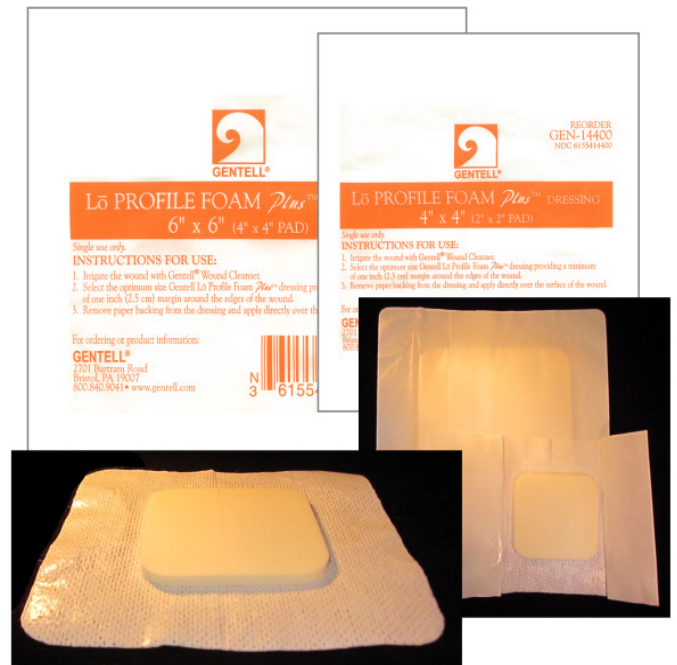

Gentell Lo-Profile Foam Plus dressings are waterproof and highly absorbent foam with an island design. Its dense, sturdy structure protects the wound from external threats, and its porous texture is highly absorbent. A non-abrasive border holds the foam in place and creates a water-resistant bacterial barrier to protect the wound.

Directions

1. Gentell Lo Profile Foam Plus may be used as a primary or secondary dressing. It may also be used as protection for wounds that have recently healed.
2. Flush the wound with Gentell Wound Cleanser and gently dry the skin surrounding the wound site.
3. Select the optimum size Gentell Lo Profile Foam Plus dressing providing a minimum of one-inch (2.5cm.) margin around the edges of the wound.
4. Remove paper backing from the dressing and apply directly over the surface of the wound.
5. Change dressing every other day or as ordered by a physician.

