



2701 Bartram Road • Bristol, PA 19007
800-840-9041 • 215-788-2700
www.gentell.com • info@gentell.com

Lo Profile Foam Plus Non-Bordered Fenestrated Dressing for Tracheostomies

- Easy to apply for tracheostomy and other tubed wound sites with moderate to heavy drainage
- Super absorptive foam keeps wounds dry to prevent maceration and reduce strikethrough
- Minimized adherence to the wound site reduces discomfort for patients

**Non-Adhesive Non-Bordered
Fenestrated Dressing**
(5"x4" pad)
50/case **GEN-14540**
NDC# 61554-145-04



Gentell's Lo-Profile Foam Plus Non Bordered Fenestrated Dressing is a pre-cut, non bordered, rectangular dressing with a U-shaped fenestration ideal for tracheostomy sites, catheters, feeding tubes or larger wound sites requiring moderate to heavy absorption. This super-absorptive Lo-Profile foam dressing reduces strikethrough and prevents skin maceration by keeping the wound site dry.

Directions

1. Gentell's Lo-Profile Foam Plus Non-Bordered Fenestrated Dressing may be cut to size and applied to pressure ulcers, chronic and acute wounds and/or partial and full thickness wounds stages 2-4.
2. Flush the wound with Gentell Wound Cleanser and gently dry the skin surrounding the wound site.
3. Place dressing over the wound and secure it in place.
4. Change dressing every other day, or when wound fluid is present at the edges, or as ordered by a physician.