



2701 Bartram Road • Bristol, PA 19007
800-840-9041 • 215-788-2700
www.gentell.com • info@gentell.com

Lo Profile Foam Plus Circular Fenestrated Dressing for Ostomies

- Easy to apply for ostomy sites, catheters and other tubed wound sites with moderate to heavy drainage
- Super absorptive foam keeps wounds dry to prevent maceration and reduce strikethrough
- Gentle, spun-lace, hypoallergenic water-resistant tape allows skin to breathe and reduces skin tears


**Adhesive Circular Fenestrated Dressing
(water-resistant backing & adhesive border)**

4" Diameter (2" Pad)

10/box 50/cs.

GEN-14410

NDC# 61554-141-04

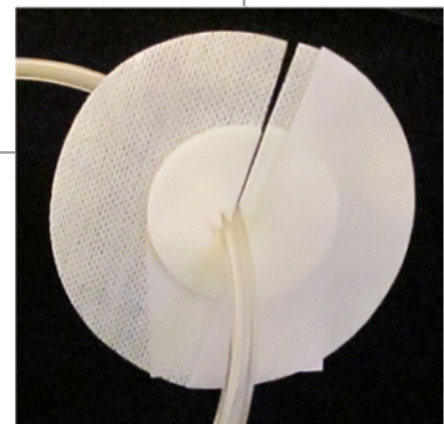

Gentell's Lo-Profile Foam Plus Circular Fenestrated Dressing is a pre-cut circular, bordered, absorptive dressing with a U-shaped fenestration ideal for ostomy sites, catheters, and feeding tubes. This super-absorptive Lo-Profile foam, dressing reduces strikethrough and prevents skin maceration by keeping the wound site dry. The hypoallergenic water-resistant tape allows the skin to breathe and makes application easy, which reduces nursing time.

Directions

1. Use Gentell's Lo-Profile Plus Circular Fenestrated dressing on pressure ulcers, chronic and acute wounds or partial and full thickness wound stages 2-4.
2. Flush the wound with Gentell Wound Cleanser and gently dry the skin surrounding the wound site.
3. Select the optimum size Gentell Lo Profile Foam Plus dressing providing a minimum of one-inch (2.5cm.) margin around the edges of the wound.
4. Remove paper backing from the dressing, and apply directly over the surface of the wound.
5. Change dressing every other day or as ordered by a physician.