



2701 Bartram Road • Bristol, PA 19007  
800-840-9041 • 215-788-2700  
www.gentell.com • info@gentell.com

# MVP Transparent Film Dressing

4" x 4.75" (10.16 cm x 12.07 cm)  
50 Dressings / Case  
**GEN-16450**  
NDC# 61554-164-1204

- Transparent dressing enables easy monitoring of wounds without removing the dressing
- Non-abrasive border enables easy application by preventing the film from rolling onto itself
- Flexible membrane conforms easily to body



**Transparent**



**Gentell MVP Transparent Film Dressing** is a moisture vapor permeable transparent membrane coated with a layer of acrylic, hypoallergenic adhesive that can be used to cover low exudating wounds. Gentell MVP is a non-absorptive sterile dressing that is permeable to moisture vapor and oxygen, but impermeable to bacteria.

## Directions

1. Gentell MVP Transparent Film Dressing can be used as a primary treatment by applying as a breathable bacterial barrier to block outside contaminants.
2. MVP can be used as a secondary dressing to Stage I or II pressure ulcers, abrasions, skin tears, blisters, skin graft donor sites, superficial partial thickness burns, autolytic debridement, skin protection against moisture as well as friction and Clean, closed surgical incisions. MVP transparent dressings may also be used to cover and secure I.V. devices.
3. Flush the wound with Gentell Wound Cleanser and gently dry the skin surrounding the wound site.
4. As a primary dressing, apply directly to the wound surface.
5. As a secondary dressing, apply directly over primary treatment.
6. Change dressing daily or as ordered by a physician.