



2701 Bartram Road • Bristol, PA 19007
800-840-9041 • 215-788-2700
www.gentell.com • info@gentell.com

FixTape

2" x 11 yards (5cm x 10 meters) 64/case GEN-10600	4" x 11 yards (10cm x 10 meters) 64/case GEN-10640 <i>(not shown)</i>
--	---

- Fixes dressings on wounds, tubes and catheters
- Two convenient sizes: 2" x 11 yards and 4" x 11 yards (not shown)
- Non-woven fabric tape with easy-to-remove self-adhesive paper backing
- Latex free



Gentell FixTape is a self-adhesive, non-woven fabric tape for the fixation of dressings, tubes and catheters with easy-to-remove protection paper. FixTape is latex free.

Directions

Open top flap. Feed tape through front edge of box. Close top flap, leaving 1 inch of tape exposed. Pull appropriate amount of tape from box. Cut tape, leaving one inch of tape exposed. Remove paper backing, and apply self-adhesive tape as required.

Warnings

- For external use only
- Non-sterile
- Store in a dry location at room temperature.