




Collagen

Particles
(1 gram)
5/box
GEN-18000

- **Reduces high levels of matrix metalloproteases (MMPs) that break down connective tissue**
- **Helpful with chronic, non-healing wounds**
- **Available in particles and multiple dressing sizes**

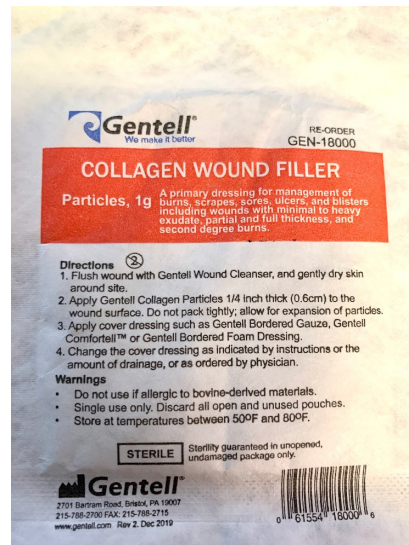

Gentell Collagen Particles are a primary dressing for management of burns, scrapes, sores, ulcers, and blisters including wounds with minimal to heavy exudate, partial and full thickness, and second-degree burns.

Directions

1. Irrigate the wound with Gentell Wound Cleanser and gently dry the skin surrounding the wound site.
2. Apply Gentell Collagen Particles directly to the wound bed. If the wound is dry (without exudate), Gentell Collagen Particles may be moistened with Gentell Wound Cleanser before applying to the wound.
3. Cover wound with a secondary dressing such as Gentell Bordered Gauze or Gentell Comfortell.
4. Repeat daily or as ordered by a physician.