



Calcium Alginate Ag (Silver) Dressing

2"x2" (5 x 5cm) 10/box GEN-13220	4"x4" (10 x 10cm) 10/box GEN-13420	4"x8" (10 x 20cm) 10/box GEN-13482	6"x6" (15.2 x 15.2cm) 10/box GEN-13620	8"x8" (20 x 20cm) 10/box GEN-13820
---	---	---	---	---

Uniformly absorbs, collects, and contains 20 times its weight in exudate with a sustained release of antimicrobial silver

- **Reduces odor primarily caused by bacteria**
- **Controlled silver release provides antimicrobial protection to the wound**



Gentell Calcium Alginate Ag (Silver) Dressings are a sterile, antimicrobial, comfortable, fiber-structured alginate with high absorbency. These advanced alginate dressings fight bacteria and a broad spectrum of microorganisms while absorbing and containing exudate in a moist healing environment. A steady release of silver and a reaction between the calcium in the dressing and the sodium in the wound exudate creates a gel that fights infection and is easy to remove from the wound. May be used on wounds, such as leg ulcers, pressure ulcers, diabetic foot ulcers, and surgical wounds or minor conditions such as lacerations, abrasions, skin tears, and minor burns.

Directions

1. Irrigate wound with Gentell Wound Cleanser and gently dry the skin around the wound.
2. Choose the appropriate size dressing based on the dimensions of the wound.
3. Cover wound with a secondary dressing such as Gentell Super Absorbent, Gentell Comfortell or Gentell Bordered Foam Dressing.
4. Repeat daily or as ordered by a physician