



Collagen

2"x2" (5 x 5cm) 10/box GEN-18200	4"x4" (10 x 10cm) 10/box GEN-18400	7"x7" (17.75 x 17.75cm) 10/box GEN-18700
---	---	---

- **Reduces high levels of matrix metalloproteases (MMPs) that break down connective tissue**
- **Helpful with chronic, non-healing wounds**
- **Available in particles and multiple dressing sizes**



Gentell Collagen is a primary dressing for management of burns, scrapes, sores, ulcers, and blisters including wounds with minimal to heavy exudate, partial and full thickness, and second-degree burns.

Directions

1. Flush wound with Gentell Dermal Wound Cleanser and gently dry skin around site.
2. Apply Gentell Collagen Wound Dressing directly on the wound bed. If the wound is dry (without exudate), Gentell Collagen may be moistened with Gentell Dermal Wound Cleanser before applying to the wound.
3. Apply cover dressing such as Gentell Bordered Gauze or Gentell Comfortell or Gentell Bordered Foam Dressing.
4. Repeat daily or as ordered by a physician.